



BIS™ SRS Semi Reusable Sensor System

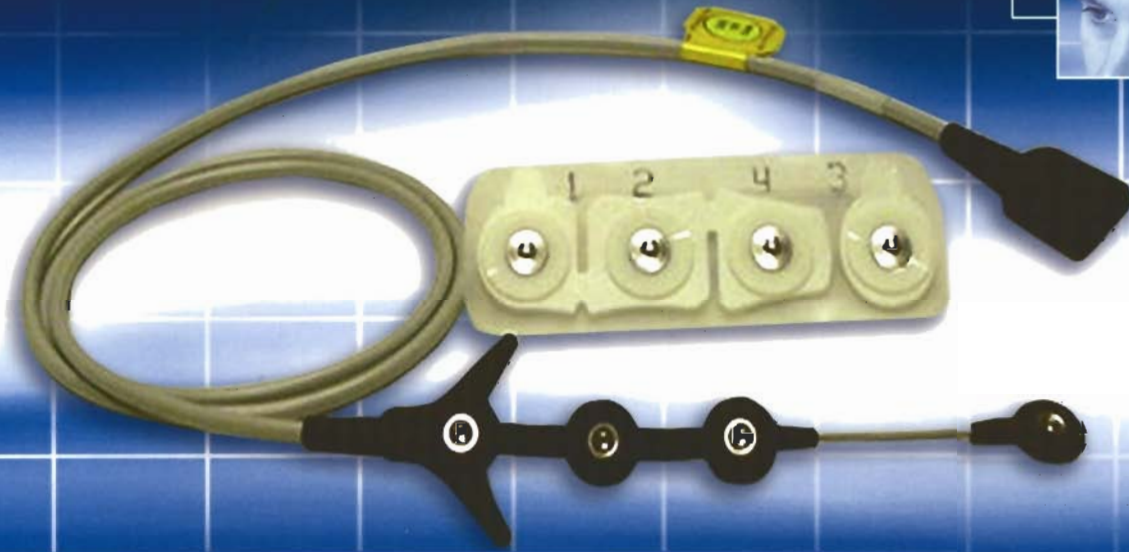

BIS SRS Semi Reusable Sensor System

- Same proven BIS™ value*
- Lower cost of ownership
- Easy to use
- Semi Reusable cable
- Disposable electrode set
- Environmentally responsible

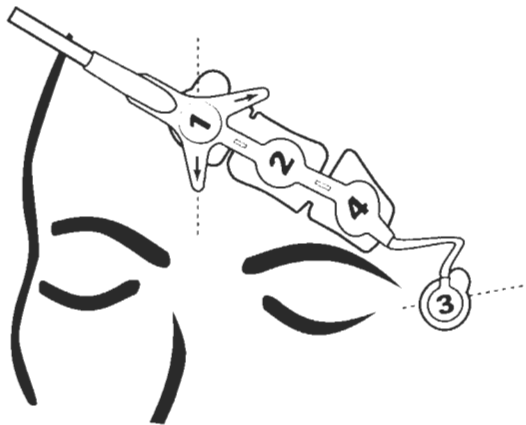

How BIS works

The BIS is a direct measure of the effects of anaesthetic and sedative agents on the brain. Using a sensor placed on the patient's forehead, BIS monitoring translates brain electrical activity into a single number between 100 (wide awake) and zero (absence of brain electrical activity) that represents the patient's level of consciousness. This enables clinicians to customise the precise type and amount of anaesthetic or sedative medication that each patient needs.

* BIS has been used on over 6 million patients and is available in more than 160 countries.



SEMI REUSABLE SENSOR SYSTEM









ASPECT®
MEDICAL SYSTEMS

BIS™ SRS Semi Reusable Sensor System



BIS SRS Dispenser

BIS SRS APPLICATION PROCEDURE

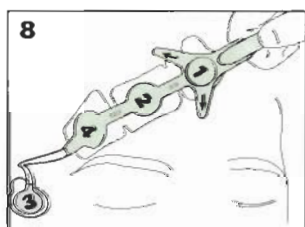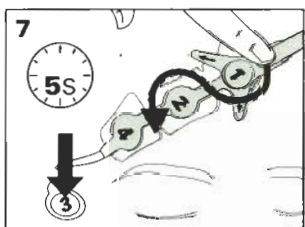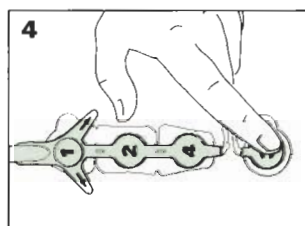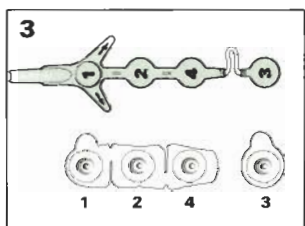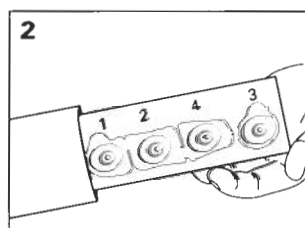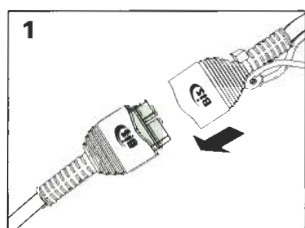

BIS SRS SEMI REUSABLE SENSOR SYSTEM PRODUCT SPECIFICATIONS

SRS DISPOSABLE ELECTRODE SENSOR

Description	4 Ag-AgCl electrodes with adhesive foam backing. Single patient use only. Not for individual resale.
Size	17.5 cm x 3.4 cm
Storage	10° C – 30° C (50° F – 85° F)
Noise RMS	0.15 – 100 Hz (<0.50uV)
AC Impedance	<500 Ohms (measured at 10 Hz, back-to-back)
Compliance	AAMI/ANSI, EC12-2000 Electrical Performance Standards

SRS Cable

Description	100 use cable housing snaps to SRS disposable electrode set. Intended for 100 uses. Not for individual resale.
-------------	--

Length	122 cm
Storage	10° C – 30° C (50° F – 85° F)

SRS Dispenser

Description	Wall mountable dispenser. Prepackaged with all system components including (1) SRS Cable and (100) SRS disposable electrode sets (4 electrodes per set)
-------------	---

Size	33.7 cm x 18.1 cm x 18.1 cm
------	-----------------------------

ORDERING INFORMATION

Part Number	Description
186-0164	BIS SRS Semi Reusable Sensor System includes: (1) SRS Cable – 100 uses (100) SRS disposable electrode sets – 4 electrodes per set

Not for sale in the USA.



Aspect Medical Systems International B.V., Rijnzathe 7d2, 3454 PV De Meern, The Netherlands.
Tel: +31 30 662 9140, Fax: +31 30 662 9150, Email: amsint@aspectms.com, Internet: www.aspectmedical.com

©2003, Aspect Medical Systems International B.V., All rights reserved. BIS and the BIS logo are trademarks of Aspect Medical Systems, Inc., and are registered in the USA, EU, and other countries. BIS Sensor U.S. Patent Number 6,032,064; 5,305,746; 6,298,255; Other Patents Pending P/N 150503-1N 3.00



SEMI REUSABLE SENSOR SYSTEM

